AT03029

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Thank you for purchasing our product. Please read the user instructions carefully. The product should not be used before reading and understanding this manual. If you do not understand the warnings, notes, and recommendations, contact a healthcare professional or the seller to avoid product damage or bodily injury.

NOTE: Check all parts for damage that may have occurred during transport. If any damage is observed, DO NOT use the product. For more information, contact the seller.

TARGET PATIENT GROUP

This product is intended for individuals with diseases, dysfunctions, or injuries that are treated, rehabilitated, or compensated by this product (see the section regarding the product's intended use in this manual). The product may be purchased by the user independently or based on recommendations from a doctor, therapist, or other specialist. Whether the product is purchased independently or recommended by a doctor/therapist/other specialist, it is important to consider the available sizes, necessary features, and variants of the product, as well as the indications, contraindications for use, and information provided by the manufacturer.

APPLICATION/INDICATIONS

Anti-pressure ulcer cushions are made from polyurethane foam in various sizes, intended for use under heels, elbows, buttocks, shoulder blades, head, knees, hips, etc. The cushions are covered with soft cotton fabric, and the internal openings of the product effectively relieve pressure on critical body points, preventing direct contact with hospital mattresses or wheelchairs. The cover is equipped with a zipper, allowing easy removal for washing. Small cushion covers (for heels, elbows, wrists) are equipped with an additional fastening strap with Velcro.

Using the product reduces the risk of pressure ulcers, but does not eliminate it entirely. To minimize the risk of pressure ulcers, regardless of product use, proper patient care is essential, particularly regular repositioning of the patient.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent safe handling of the product.

Allergies to materials used in the product.

Skin damage, burns, etc.

COMPONENTS OF THE PRODUCT

Anti-pressure ulcer cushion, user manual

FEATURES AND SIZES OF THE PRODUCT

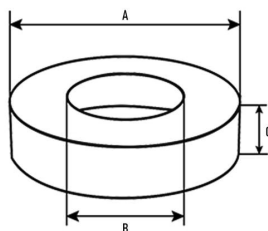
Polyurethane foam

The cover is made of cotton terry fabric (velour) in navy blue color

The covers for small cushions (1,2) are equipped with an additional fastening strap with Velcro.

The cushions are available in the following dimensions (A x B x C):

1. $\varnothing 15 \times \varnothing 7 \times 5$ [cm]
2. $\varnothing 20 \times \varnothing 10 \times 6$ [cm]
3. $\varnothing 40 \times \varnothing 15 \times 5$ [cm]

APPLICATION OF ANTI-PRESSURE ULCER CUSHIONS

To help choose the appropriate anti-pressure ulcer cushion, the suggested use of the different sizes is as follows:

Cushion Ø15 x Ø7 x 5 cm – recommended for relieving pressure on heels, elbows, and wrists.

Cushion Ø20 x Ø10 x 6 cm – suitable for supporting the head and lower limbs.

Cushion Ø40 x Ø15 x 5 cm – designed for relieving pressure on the coccyx.

The indications provided are for guidance only. The choice of the appropriate model should take into account the user's individual needs and medical recommendations.

USAGE

Preparing the cushion for use:

Ensure that the cushion is clean and dry.

Check that the cover is properly fitted and does not have any visible damage.

Placement of the cushion:

Gently lift the body part you wish to relieve pressure from.

Place the cushion so that the area at risk of pressure sores is positioned directly above the hole in the cushion, ensuring that pressure is relieved from this area.

Anti-pressure ulcer cushions with dimensions Ø15 x Ø7 x 5 cm and Ø20 x Ø10 x 6 cm are equipped with additional fastening straps. After placing the cushion under the relieving body part, fasten the strap to ensure stabilization of the limb.

WARNING:

In the event of a serious incident related to this product, which directly or indirectly led, could have led, or may lead to any of the following events:

- a) the death of the patient, user, or any other person, or
- b) temporary or permanent deterioration of the health of the patient, user, or any other person, or
- c) a serious threat to public health,

the above serious incident must be reported to the manufacturer and the competent authority of the country of residence of the user or patient.

WARNING:

If pain, allergic reactions, or any other disturbing or unclear symptoms occur related to the use of the medical device, consult a healthcare professional.

WARNING:

It is prohibited to use the product in any way other than as intended.

CLEANING

Cleaning the Cover:

Hand wash at a maximum temperature of 40°C

Do not iron

Do not dry clean

Do not dry in a tumble dryer

Do not bleach

Cleaning the Foam:

WARNING: Do not wash the foam in a washing machine!

Cleaning the foam requires a gentle approach to avoid damaging its structure. Here is a guide on how to clean it properly:

What you will need:

Mild detergent or dishwashing liquid

Warm water

Baking soda (optional for deodorizing)

Vacuum cleaner (preferably with a soft brush)

Spray bottle

Soft cloth or sponge

Enzymatic cleaner (optional for stain removal)

Cleaning Steps:**Vacuuming the foam:**

Remove dust and debris: Use a vacuum cleaner with a soft brush to gently vacuum the foam surface. This will remove dust, dirt, and contaminants before wet cleaning.

Spot cleaning with mild detergent:

Mix detergent: Combine a few drops of mild detergent with warm water in a spray bottle.

Gently spray the foam: Avoid soaking the foam. Lightly spray the stained area with the solution.

Blot the stain: Use a soft cloth or sponge to gently blot the area. Do not rub, as this could damage the foam structure.

Rinse the area: If necessary, lightly spray clean water to remove any detergent residue.

Dealing with stubborn stains:

Biological or urine stains: Use an enzymatic cleaner. Lightly apply the cleaner to the stain, leave it for a while (as instructed by the product), and then blot it with a clean cloth.

Grease stains: Use a small amount of dishwashing liquid or a special upholstery cleaner designed for foam materials.

Deodorizing (optional):

Use baking soda: Sprinkle a layer of baking soda on the foam and leave it for several hours or overnight to absorb odors.

Vacuum the baking soda after the recommended time.

Drying the foam:

Dry completely in the air: The foam is highly absorbent, so it may take a while to dry. Leave it in a well-ventilated area.

Use a fan to speed up the process: Never use direct heat sources, such as a hairdryer, as this may damage the foam. It is best to use a cool fan.

Ensure the foam is completely dry before reuse to prevent mold or mildew growth.

What to avoid:

Never submerge the foam in water, as it may be difficult to dry and may get damaged.

Avoid using harsh chemicals like bleach or alcohol-based cleaners, as they can weaken the foam structure.

No direct heat: Heat can damage the foam structure, so avoid drying in a tumble dryer or using hot air.

STORAGE

Store the product in a dry place, away from direct sunlight.

DISPOSAL

After the product is no longer in use, dispose of it as regular household waste. Follow waste segregation regulations and protect the environment.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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